TANITA

Monitoring Your Health

Professional Raised Dial


Operating Instructions – HA-502

OPERATING INSTRUCTIONS

Professional Raised Rotating Indicator Scale – HA-502

- ① Measuring Platform
- ② Dial Display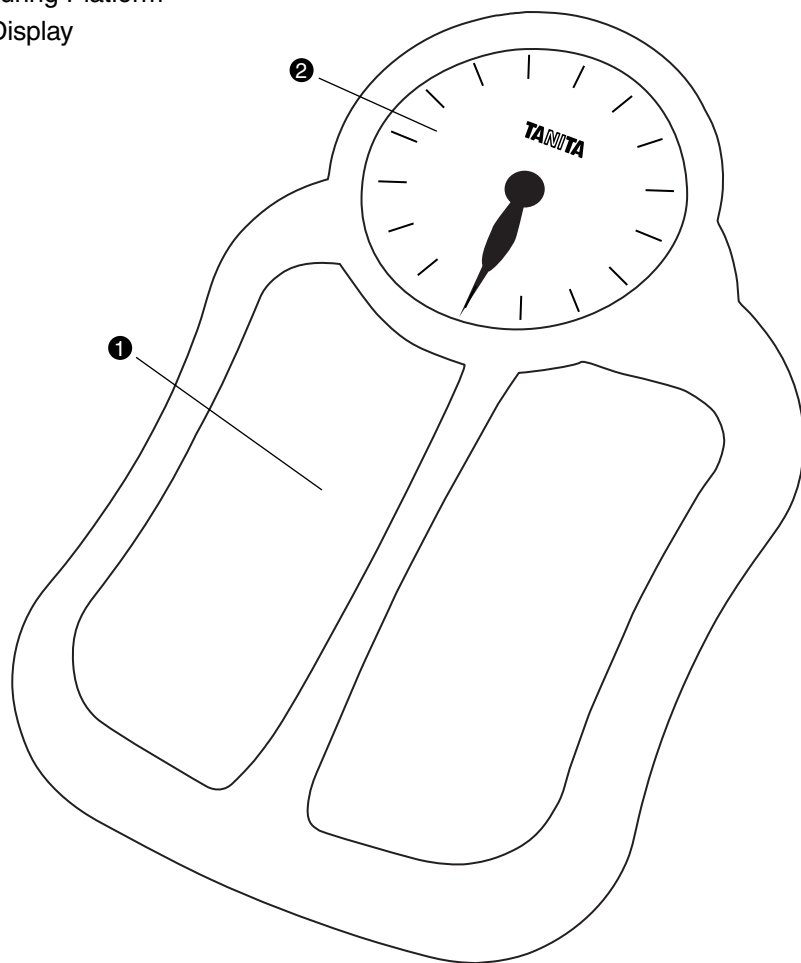

Specifications

Maximum Capacity: 400 lb (180 kg)

Graduation: 1 lb (0.5 kg)

UK Specifications

Maximum Capacity: 23 st (150 kg)

Graduation: 1 st (1 kg)

Features

- 7" raised white pearl dial
- Accuracy and reliability.
- Fits any décor.
- Durable construction, non-skid surface.

Placing Scale

The scale should be used on a hard even surface for accurate weighing. Not recommended for use on carpeted surfaces. Placing scale on a very soft, deep pile rug may affect weighing accuracy.

Instructions for Use

Before using scale, allow it to reach room temperature; then put scale through several weighing cycles consisting of:

Step on scale and allow dial to come to rest.

Step off scale and allow dial to come to rest.

After weighing cycles are completed, adjust dial to zero, if necessary (adjustment knob under edge of platform), and repeat weighing cycles until zero reading is stabilized.

If scale is relocated, readjustment of zero may be required. Repeat above procedure.

When weighing, place feet on platform so weight is centralized.

Place on hard, level floor surface.

Care and Maintenance

An occasional wiping with a soft cloth and mild soap will keep your scale attractive. Do not use abrasive cleaners. Your scale does not require oiling or lubrication of any kind.

After Sales Service

Please contact the Tanita office nearest you regarding operation or trouble, request for repair, and/or placement of an order for parts for products.

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Tanita France S.A.
Tanita Health Equipment H.K. Ltd.
Tanita Europe GmbH
Tanita UK Ltd.