

TANITA

Monitoring Your Health


BD-585

Digital Baby Scale


Báscula (Pesa) Digital para Bebés

Instruction Manual

Manual de Instrucciones

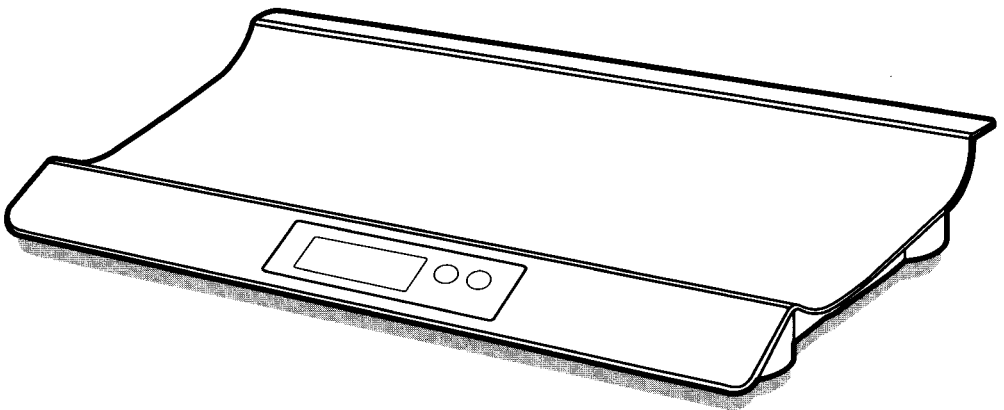

Components and Features

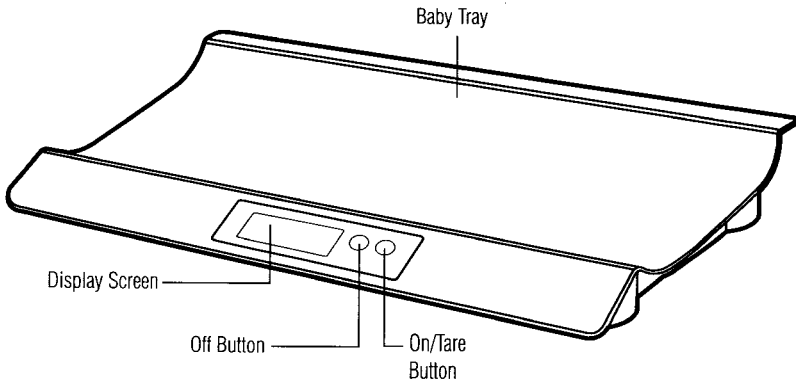


Preparations Before Use



Set scale to measure in pounds or kilograms; use the switch located on the bottom of the scale.

Inserting the Batteries

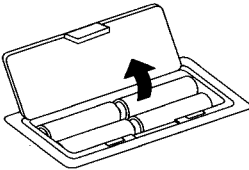


Locate the battery cover on the bottom of the scale. Press the tab to release the cover. Insert 4 AA alkaline batteries as indicated.

Note: When you insert the battery, make sure that the negative end touches flat to the spring.

Note: For best results, use alkaline batteries. Do not mix alkaline and manganese batteries together. Avoid using rechargeable batteries.

Important Notes

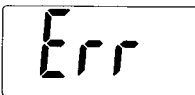

- Do not get the scale wet or store in a humid place.
- The scale is a precision instrument—do not drop it, jump on or drop heavy objects onto it.
- Do not dismantle or adjust the scale.
- When pressing the On/Tare button, do not hold it down. This will cause the "Err" (error) message to appear on the display.

Taking Weight Measurement



ON/TARE

Turn on the power by pressing the On/Tare button.
(The display will flash all items.)



"88888" will appear on the display.



Wait for the display to read "0 0.0" and place the baby on the scale (within 60 seconds).



Display will show weight.



After taking the baby away from the tray and after a few seconds, the display will return to "0 0.0" and turn off automatically.

Tare Function

The Tare function allows you to weigh the baby with another item (i.e. towel or blanket) without skewing the actual weight of the baby.

Turn on scale.

Place item (i.e. towel or blanket) on the baby tray.


Press the On/Tare button to reset scale to "0 0.0".

Place baby on the scale.



Note: When the total weight of item and baby's weight exceeds 40 lb or 20 kg, the display shows "OL" (overload).

Replacing Batteries

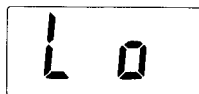

If "Lo" appears on the display, batteries are running down and need to be replaced. If nothing appears on the display the batteries are dead and need to be replaced.

Specifications

Weight Capacity

40 lb/20 kg

Weight Graduation

0.5 oz (0–40 lb)

10 g (0–20 kg)

Power Supply

4 x AA batteries (included)

Display

1"/26 mm

Color

Pure White

UPC No.

742496-858504

Detalles Técnicos

Capacidad de Peso

40 libras / 20 kg

Graduación de Peso

0.5 onza (0–40 lb)

10 g (0–20 kg)

Suministro de Energía

4 baterías de tamaño AA (que se incluyen)

Pantalla

1 pulgada / 26 mm

Color

Blanco Puro

No. UPC

742496-858504

TANITA

Monitoring Your Health

www.tanita.com

Tanita Corporation

14-2, 1-Chome, Maeno-Cho
Itabashi-Ku, Tokyo, Japan 174-8630

Tel: 03-3968-2123

Fax: 03-3967-3766

ISO 9001 Certified

Tanita France S.A.

Villa Labrouste
68 Boulevard Bourdon
92200 Neuilly-Sur-Seine, France

Tel: 01 55 24 99 99

Fax: 01 55 24 98 68

Tanita Europe GmbH

Dresdener Strasse 25
D-71065 Sindelfingen, Germany

Tel: 07031-6189-6

Fax: 07031-6189-71

Tanita Corporation of America Inc.

2625 South Clearbrook Drive
Arlington Heights, IL 60005 USA

Tel: 847-640-9241

Fax: 847-640-9261

Toll Free: 1-877-6-TANITA

Tanita Health Equipment H.K. Ltd.

Unit 301-303 3/F Wing On Plaza
62 Mody Road, Tsimshatsui East

Kowloon, Hong Kong

Tel: 852-2838-7111

Fax: 852-2838-8667

Tanita UK Ltd.

The Barn, Philpots Close, Yiewsley,
West Drayton, Middlesex, UB7 7RY, UK

Tel: 01895-438577

Fax: 01895-438511