

## D-1000plus & D-1100plus Quick Set-Up Guide

You will receive readings only if data has been programmed into the Tanita Remote Display.

### 1. Initial Setting (Clock Setting)

Insert batteries into the remote display and then press any front panel button to set the clock.

#### 1) Set Hour

Press ▲/▼ buttons to set hour and then press SET. The unit will beep once to confirm.

#### 2) Set Minute

Press ▲/▼ buttons to set minute and then press SET. The unit will beep twice to confirm the programming.

#### • Resetting Clock


Press the clock button on the bottom of the remote display. Then follow the clock setting procedure 1) and 2).



### 2. Pairing the remote display and Tanita wireless platform ("platform")

The platform needs to pair with the applicable remote display before measurements are attempted.

**Note: If there is no serial number on the bottom of the platform, the pairing process is not required and you can skip to the section 3, setting and storing personal data in the remote display.**

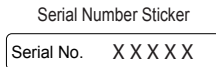

- 1) Before proceeding,
  - a) Install the batteries in the platform.
  - b) Install the batteries and set the clock in the remote display.
  - c) The platform and the remote display are within 17 feet (5 m) of each other.

- 2) Press and hold the OFF button for 2 seconds on the remote display. The remote display will show "1234" for selecting a personal number. Select the preferred personal number which will be paired with the platform using ▲/▼ buttons and press the SET button to confirm.



- 3) The remote display will show "SEt-P". Select "YES" or "no" using ▲/▼ buttons, and press the SET button to confirm,
  - a) "YES": The pairing process will continue to the next step.
  - b) "no": The pairing process will be cancelled and then the clock will be displayed.



- 4) Input the 5 digit serial number found on the bottom of the platform using ▲/▼ buttons and press the SET button to confirm.




Cancellation-1: Press the OFF button to cancel during the pairing process and then the clock will be displayed.  
Cancellation-2: The pairing process will be cancelled if inactive for 1 minute and then the clock will be displayed.

**Note:** • "00000", "000001" and more than "65535".

- These serial numbers are not allowed. Please input the correct serial number.
- The serial number is located on the bottom of the platform.

## D-1000plus & D-1100plus Quick Set-Up Guide

- 5) Once the pairing process is completed with the platform, the remote display shows "donE" and then the clock will be displayed. If the remote display shows "P-Err", please try the pairing process again.

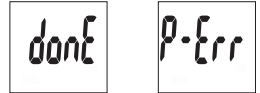

#### Note:

**To confirm the paired serial number on each personal number after completion the pairing process:**

Press and hold the SET button for 2 seconds on the remote display. The remote display will show "-1-". Select the personal number using ▲/▼ buttons, and press the SET button to confirm. The display shows the paired serial number for 3 seconds.

"- - - -" is shown if there is no serial number stored for the selected personal number.

### 3. Setting and storing personal data in the remote display

You will receive readings only if data has been programmed into the Tanita remote display.

#### 1) Selal Data Number

Use the SET button to activate the unit. Use ▲/▼ buttons to select a personal data number. Press the SET button.

#### 2) Set Age (Applicable Age: Refer to the platform manual)

Use ▲/▼ buttons to select age. Press the SET button.

#### 3) Select Female or Male (Athlete mode: Refer to the platform manual)

Use ▲/▼ buttons to select Female / Male / Athlete Female / Athlete Male. Press the SET button.

#### 4) Specify Height

Use ▲/▼ buttons to select height. Press the SET button to confirm.

#### 5) Select Activity Level

**Note:** D-1000plus only, but it also depends on the platform specification, refer to platform manual.

Use the ▲/▼ buttons to specify Activity Level. Press the SET button to confirm.

The remote display will beep twice and the display will flash all data three times to confirm the programming. The power will then shut down automatically.

To use the DCI (Daily Calorie Intake) feature properly, you need to set the Activity Level.

Level 1 = Inactive lifestyles (little or no exercise)

Level 2 = Moderately active lifestyles (occasional, low intensity exercise)

Level 3 = Adult involved in intense physical activity

**Note:** If you select the Athlete Mode, it will automatically set the Activity Level to Level 3.

Please review the Tanita wireless platform instruction manual for remote display operations, warnings, and other important information. The instruction manual of each platform also includes the following procedures.

- Taking body composition readings
- Recall Function
- Programming The Guest Mode
- Taking Weight Measurement Only

**For questions or help with product:** Call Customer Service (847) 640 - 9241 (US & Canada Only)



Do not use the body fat reading feature of this product if you have a pacemaker or other electronic implanted medical device. The body fat monitor function is not intended for pregnant women, professional athlete or bodybuilders.